

## DRI<sup>®</sup> Amphetamine Cross-Reactivity Tables Test Code # 500/500Q

### POSITIVE COMPOUNDS

Concentrations of compounds tested that produced a result approximately equivalent to the cutoff calibrator (1000 ng/mL)

Positive Compounds	Trade Name	Concentration Tested (ng/mL)
d-Amphetamine		1.0
d-Methamphetamine		1.0
Methylenedioxyamphetamine (MDA)		2.5
Methylenedioxymethamphetamine (MDMA)		1.3
Isometheptene	Midrin, Duradrin	16
Tyramine		500

### NEGATIVE COMPOUNDS

Concentrations of compounds tested that produced a negative result relative to the cutoff calibrator (1000 ng/mL)

Negative Compounds	Trade Name	Concentration Tested (µg/mL)
Acetaminophen	Anacin, Datri Extra, Liquiprin, Panadol Tempra, Tylenol	1,000
Acetylsalicylic Acid	Aspirin, Bufferin	1,000
l-Amphetamine		12.5
Benzoylcegonine		1,000
Benzphetamine	Didrex	20
Benzylpiperazine		63
Bupropion	Wellbutrin, Zyban	50
Caffeine	Coffee Break, Durivitan, No-Doz, Percofedrinol N, Pro-Plus, Vivarin	1,000
Cetirizin Dihydrochloride	Zyrtec	1,000
Chlorpromazine	Ormazine, Thorazine	500
Codeine		1,000
Dextromethorpan	Benylin DM, Delsym, Hold, Perfussin 8hr, Mediquell, Sucrets	1,000
d-Ephedrine		2,000
d, l-Ephedrine		700
l-Ephedrine		350

Negative Compounds	Trade Name	Concentration Tested (µg/mL)
Fenfluramine	Pondimin	4.0
Isoxsuprine	Vasodilan, Voxsuprine	100
Meperidine	Demerol	1,000
Mephentermine	Wyamine	25
Methadone	Dolophine	1,000
l-Methamphetamine		10
Methapyrilene	Ludiomil	500
Methylphenidte	Ritalin	500
Morphine		1,000
Nor-pseudoephedrine		1,000
Oxazepam	Serax	500
Phencyclidine		1,000
Phendimetrazine	Bontril, Plegine	200
Phenethylamine		100
Phenmetrazine	Preludin	4.0
Phenobarbital	Luminal	1,000
Phenothiazine		10
Phentermine	Adipex, Fastin, Ionamin, Obenix, Obephen, Obermine, Obestin, Phentamine	25
Phenylephrine	Alconeprin, Neo-Synephrine, Rhinall, Sinarest, Vicks Sinex,	500
Phenylpropanolamine		250
Procainamide		20
Promethazine	Phenergan	500
Propranolol	Betachron, Inderal	200
d-Pseudoephedrine		250
l-Pseudoephedrine		500
Ranitidine	Zantac	600
Scopolamine	Transderm-Scop	100
Secobarbital	Seconal	1,000
Thiorizadine	Mellaril	1,000
Trifluoperazine	Ultram	1,000
3-OH-Tyramine		500