

DRI[®] Benzodiazepine Cross-Reactivity Tables
Test Code # 502/502Q

POSITIVE COMPOUNDS

The following compounds tested POSITIVE using the DRI[®] DAU Benzodiazepine assay at the 200 ng/dL cutoff.

Positive Compounds	Trade Name	Concentration Tested (ng/mL)
Alprazolam	Tafil, Valeans, Xanax, Xandor	105
7-Aminoclonazepam		2500
Bromazepam	Lectopam	225
Chlordiazepoxide	Libritabs, Librium, Sereen	1100
Clobazam		145
Clonazepam	Klonopin	500
Clorazepate	Gen-XENE, Tranxene	120
Delorazepam		110
Desmethyldiazepam (nor-diazepam)	Nordaz, Calmday	100
Diazepam	Valium, Valrelease, Vazepam	95
Flunitrazepam	Rohypnol	175
Flurazepam	Dalmane, Durapam	140
Lorazepam	Ativan	1,000
Lormetazepam		225
Medazepam	Nobrium	225
Midazolam		900
Nitrazepam		175
Norfludiazepam		115
Prazepam	Centrax	110
Temazepam	Restoril, Razepam, Temaz	125
Triazolam	Halcion	125

NEGATIVE COMPOUNDS

The following compounds tested **NEGATIVE** on the DRI[®] DAU Benzodiazepine assay at the 200 ng/mL cutoff.

Negative Compounds	Trade Name	Concentration Tested (µg/mL)
Acetaminophen	Anacin, Datri Extra, Liquiprin, Panadol Tempra, Tylenol	1,000
Acetylsalicylic Acid	Aspirin, Bufferin	1,000
Amphetamine		
Caffeine	Coffee-Break, Durivitan, No-Doz, Percoffedrinol N, Pro-Plus, Vivarin	100
Codeine		1,000
Dextromethorphan	Benylin DM, Delsym, Hold, Perfussin 8 hr, Mediquell, Sucrets	1,000
Diphenhydramine	Banophen, Benadryl, Diphedryl, Simply Sleep	500
Fluoxetine		500
Gabapentin	Neurontin	1,000
Methadone	Dolophine	1,000
Morphine		200
Nor-Fluoxetine		500
Nor-Sertraline		1,000
Oxaproxin	Daypro	50
Paroxetine		500
Phencyclidine		1,000
Propoxyphene	Darvor	1,000
Secobarbital	Seconal	1,000
Sertraline		500