

CEDIA[®] Buprenorphine Cross-Reactivity Tables
Test Code # 521/521Q

POSITIVE COMPOUNDS

The following compounds tested POSITIVE on the CEDIA[®] DAU Buprenorphine assay at the 5 ng/mL cutoff.

Positive Compounds	Trade Name	Concentration Tested (ng/mL)
Amisulpride		31,125
Buprenorphine	Buprenex, Subutex	5
Buprenorphine-3-b-d-glucuronide		5
Codeine		100,000
Dihydrocodeine	DHC Plus, Synalgos-DC	100,000
Hydrocodone	Lortab, Vicodin	100,000
Nalbuphine	Nubian	250,000
Nalorphine		100,000
Naltrexone	Depade, ReVia	100,000
Norpropoxyphene		100,000
Quinine		31,250
Tramadol	Ultram	150,000

NEGATIVE COMPOUNDS

The following compounds tested NEGATIVE on the DRI[®] DAU Buprenorphine assay at the 300 ng/mL cutoff.

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
6-Monoacetylmorphine		100,000
10, 11-Dihydrocarbamazepine		500,000
11-nor-THC-COOH		500,000
Acetaminophen (paracetamol)	Anacin, Datri Extra, Liquiprin, Panadol Tempra, Tylenol	500,000
Acetylsalicylic Acid	Aspirin, Bufferin	500,000
Amitriptyline	Elavil, Endep, Enovil, Emitrip	500,000
Amoxicillin	Amoxil, Polymor, Trimox, Wymox	500,000
Amphetamine		500,000
Benzotropine mesylate	Cogentin	500,000
Benzoylcegonine		500,000
Brompheniramine	Dimetane, Dimetapp, Nasahist, ND-Stat Oraminic II	500,000

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Caffeine	Coffee-Break, Durivitan, No-Doz, Percoffedrinol N, Pro-Plus, Vivarin	500,000
Captopril	Capoten	500,000
Chlordiazepoxide	Libritabs, Librium, Sereen	500,000
Cimetidine	Tagamet	500,000
Codeine-6-glucoronide		100,000
Desipramine	Norpramin, Pertofrane	500,000
Dextromethorphan HBr	Benilyn DM, Delsym, Hold, Pefussin 8 hr, Mediquell, Sucrets	100,000
Diacetylmorphine	Heroin	100,000
Diazepam	Valium, Valrelease, Vazepam	500,000
Digoxin	Lanoxin	500,000
Diphenhydramine	Banophen, Benadryl, Diphedryl, Simply Sleep	500,000
Disopyramide		500,000
Doxepine	Adapin, Sinequan	500,000
Doxylamine	Decapryn, Nighttime Sleep Aid, Unisom	500,000
Doxycycline Hyclate	Periostat	500,000
EDDP (Primary Methadone Metabolite)		100,000
EMDP (Secondary Methadone Metabolite)		100,000
Enalapril	Vasotec	500,000
Fluoxetine	Prozac, Sarafem	500,000
Haloperidol	Haldol, Peridol	500,000
Heroin		100,000
Hydromorphone	Dilaudid	100,000
Ibuprofen	Advil, Haltran, Mediprene, Motrin, Nuprin, Proen, Rufen	500,000
Imipramine	Tofranil	100,000
LAAM		100,000
a-Levoacetylmetadol		100,000
a-Levodonoracetylmetadol		100,000
a-Levonoracetylmetadol		100,000
Levorphanol		100,000
Levothyroxine	Synthroid	500,000
Meperidine	Demerol	100,000
a-Methadol		100,000
Methadol		100,000
Methadone	Dolophine	100,000

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Methamphetamine	Desoxyn	500,000
Morphine	Nubain	100,000
Morphine-3-glucuronide		100,000
Morphine-6-glucuronide		100,000
Naloxone	Narcan	100,000
Nifedipine	Procardia	500,000
Norbuprenorphine		1,000
Norbuprenorphine-3-b-d-glucuronide		1,000
Nordiazepam (Desmethyldiazepam)		500,000
Noroxycodone		100,000
Noroxymorphone		100,000
Oxazepam	Serax	500,000
Oxycodone	OxyContin	100,000
Oxymorphone	Numorphan	100,000
Pentazocine	Talwin	500,000
Pherphenazine	Trilafon	500,000
Phencyclidine (PCP)		500,000
Phenobarbital	Luminal	500,000
Procyclidine	Kemadrin	500,000
Propoxyphene		500,000
Protriptyline	Vivactil	500,000
Ranitidine	Zantac	500,000
Salicylic acid		500,000
Secobarbital	Seconal	500,000
Tolmetin	Tolectin	500,000
Tripolidine	Actagen, Allergin, Aprodine	500,000
Verapamil	Isoptin, Verelan	500,000