

## DRI<sup>®</sup> Cocaine Cross-Reactivity Tables Test Code # 503/503Q

### POSITIVE COMPOUNDS

The following compounds tested POSITIVE using the DRI<sup>®</sup> DAU Cocaine assay at the 300 ng/mL cutoff.

Positive Compounds	Trade Name	Concentration Tested (ng/mL)
Benzoyllecgonine		.03
Cocaine		50
Ecgonine		100

### NEGATIVE COMPOUNDS

The following compounds tested NEGATIVE on the DRI<sup>®</sup> DAU Cocaine assay at the 300 ng/mL cutoff.

Negative Compounds	Trade Name	Concentration Tested (µg/mL)
Acetaminophen	Anacin, Datri Extra, Liquiprin, Panadol Tempra, Tylenol	1,000
Acetylsalicylic Acid	Aspirin, Bufferin	1,000
Acyclovir	Zovirax	100
Albuterol	Proventil, Ventolin, Salbutamol	1,000
Amikacin	Amikin	1,000
Amitriptylline	Elavil, Endep, Enovil, Emitrip	100
Amobarbital	Amytal	1,000
Amoxicillin	Amoxil	1,000
Amphetamine		1,000
Azithromycin	Zithromax	100
Benzocaine		1,000
Benzoyllecgonine		1,000
Buprenorphine	Buprenex	10
Bupropion	Wellbutrin, Zyban	100
Caffeine	Coffee Break, Durivitan, No-Doz, Percofedrinol N, Pro-Plus, Vivarin	100
Calcium Carbonate	Oscal	5,000
Chlorpromazine	Ormazine, Thorazine	500
Clonidine	Catapress	100

Negative Compounds	Trade Name	Concentration Tested (µg/mL)
Cocaethylene		100
Codeine		1,000
Dapsone	Avlosulfon	10
o-Desmethylvenlafaxine		1,000
Dextromethorpan	Benylin DM, Delsym, Hold, Perfussin 8hr, Mediquell, Sucrets	100
Diphenhydramine	Banophen, Benadryl, Diphedryl, Simply Sleep	1,000
Doxepin	Adapin, Sinequan	500
Doxycycline Hyclate	Periostat	100
Ecgonine Methyl Ester		100
Fentanyl	Duragesic, Actiq, Sublimaze	10
Fluconazole	Diflucan	100
Fluoxetine	Prozac, Sarafem	50
Gabapentin	Neurontin	100
Gentamicin	Garamycin	1,000
Hydroxyzine	Anxanil, Apo-Hydroxyzine, Atarax, Hyzine-50	100
Hyoscyamine HCl	Anaspaz, Levsin	100
Ibuprofen	Advil, Haltran, Mediprene, Motrin, Nuprin, Proen, Rufen	5,000
Indomethacin	Novomethacin, Indocin	100
Lamivudine	Epivir	100
Lamotrigene	Lamictal	1,000
Levofloxacin	Levaquin	100
Lidocaine	Dilocaine, LidoPen, Solarcaine, Xylocaine	1,000
Lithium heparin		5,000
Loratadine	Claritin	500
Meperidine	Demerol	1,000
Mesoridazine	Serentil	1,000
Methadone	Dolophine	1,000
Methylphenidate	Ritalin	100
Metoclopramide	Clopra, Maxolon, Octamide, Reglan	1,000
Metronidazole	Flagyl	100
Morphine		200
Nalbuphine	Nubain	1,000
Naltrexone	Depade, ReVia	1,000
Naproxen Na <sup>+</sup>	Aleve, Anaprox, Naprosyn	5,000
Norfluoxetine HCl		1,000

Negative Compounds	Trade Name	Concentration Tested (µg/mL)
Oflaxacin	Floxin	100
Omeprazole	Prilosec	1,000
Oxazepam	Serax	100
Paroxetine	Paxil	100
Phencyclidine		1,000
Phenelzine	Nardil, Parnate, Marplan	100
Phenobarbital	Luminal	1,000
Phenytoin (DPH)		500
Pholcodine		3.9
Promethazine	Phenergan	100
Propoxyphene	Darvon	1,000
Ranitidine	Zantac	100
Risperidone	Risperdal	100
Scopolamine	Transderm-Scop	1,000
Secobarbital	Seconal	1,000
Spirolactone	Novospiroton	1,000
Stavudine	Epivir	1.0
Thiopental	Pentothal Sodium	1,000
Thiorizadine	Mellaril	1,000
Tobramycin	Nebcin, Tobi, Tobrex	1,000
Tolmetin	Tolectin	1,000
Tramadol	Ultram	500
Trazodone	Desyrel	1,000
Trimethoprim	Proloprim, Trimprex	5,000
Vancomycin	Vancocin	1,000
Venlafaxine	Effexor	1,000