

DRI[®] Ecstasy Cross-Reactivity Tables Test Code # 511/511Q

POSITIVE COMPOUNDS

The following compounds tested POSITIVE using the DRI[®] DAU Ecstasy assay at the 500 ng/mL cutoff.

Positive Compounds	Trade Name	Concentration Tested (ng/mL)
Fenofibrate	Tricor	80,000
MDA		900
MDEA		600
MBDB		2,000
BDB		1,500
para-Methoxyamphetamine (PMA)		4,700
para-Methoxymethamphetamine (PMMA)		1,700

NEGATIVE COMPOUNDS

The following compounds tested NEGATIVE on the DRI[®] DAU Ecstasy assay at the 500 ng/mL cutoff.

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
d. l-Amphetamine		180,000
d-Amphetamine		600,000
l-Amphetamine		90,000
Acetaminophen	Anacin, Datri Extra, Liquiprin, Panadol Tempra, Tylenol	1,000,000
Acetylsalicylic Acid	Aspirin, Bufferin	1,000,000
Amoxicillin	Amoxil	1,000,000
Benzoyllecgonine		400,000
Bupropion	Wellbutrin, Zyban	100,000
Caffeine	Coffee Break, Durivitan, No-Doz, Percofedrinol N, Pro-Plus, Vivarin	100,000
Captopril		1,000,000
Chlordiazepoxide	Libritabs, Librium, Sereen	250,000
Cimetidine	Tagamet	500,000
Codeine		1,000,000
Diazepam	Valium	1,000,000

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Digoxin	Lanoxin	100,000
Enalapril	Vasotec	800,000
Fluoxetine	Prozac, Sarafem	500,000
Ibuprofen	Advil, Haltran, Mediprene, Motrin, Nuprin, Proen, Rufen	1,000,000
Levothyroxine	Synthroid	100,000
Methadone	Dolophine	1,000,000
d, l-Methamphetamine		100,000
d-Methamphetamine		62,000
Morphine	MSIR, Ms Contin, Oramorph Sr, Roxanol, Astramorph, Duramorph, Infumorth	1,000,000
Nifedipine	Procardia	50,000
Phencyclidine		1,000,000
Phenmetrazine		1,000,000
Phenobarbital	Luminal	1,000,000
Phentermine	Adipex, Fastin, Ionamin, Obenix, Obephen, Obermine, Obermine, Obestin, Phentamine	220,000
d, l-Phenylpropanolamine	Acutrim, Dexatrim, Dura-Vent	800,000
Propoxyphene	Darvon	1,000,000
d-Pseudoephedrine		1,000,000
Salicylic Acid		1,000,000
THC		10,000
Tolmetin	Tolectin	500,000
Verapamil	Calan, Isoptin, Verelan	1,000,000