

DRI[®] Methadone Metabolite Cross-Reactivity Tables
Test Code # 504/504Q

POSITIVE COMPOUNDS

The following compounds tested POSITIVE using the DRI[®] DAU Methadone Metabolite assay at the 1000 ng/mL cutoff.

Positive Compounds	Trade Name	Concentration Tested (ng/mL)
Methadone Metabolite (EDDP)		1,000

NEGATIVE COMPOUNDS

The following compounds tested NEGATIVE using the DRI[®] DAU Methadone Metabolite assay at the 1000 ng/mL cutoff.

Negative Compounds	Trade Name	Concentration Tested (µg/mL)
11-nor- Δ^9 -THC-9-COOH		10,000
Acetaminophen	Anacin, Datri Extra, Liquiprin, Panadol, Tempra, Tylenol	1,000,000
Acetylsalicylic Acid	Aspirin, Bufferin	1,000,000
Amphetamine		1,000,000
Benzoyllecgonine		1,000,000
Caffeine	Coffee Break, Durivitan, No-Doz, Percoffedrinol N, Pro-Plus, Vivarin	100,000
Captopril	Capoten	500,000
Chlordiazepoxide	Libritabs, Librium, Sereen	100,000
Cimetidine	Tagamet	500,000
Cocaine		200,000
Codeine		1,000,000
Dextromethorphan	Benylin DM, Delsym, Hold, Perfussin 8 hr, Mediquell, Sucrets	300,000
Diazepam	Valium, Valrelease, Vazepam	100,000
Diphenhydramine	Banophen, Benadryl, Diphedryl, Simply Sleep	500,000
Disopyramide		1,000,000
Doxylamine	Decaprin, Nighttime Sleep Aid, Unisom	500,000
EMDP (Secondary Methadone Metabolite)		200,000

Negative Compounds	Trade Name	Concentration Tested (µg/mL)
Ephedrine	Kondon's Nasal, Pretz-D	1,000,000
Fluoxetine	Prozac, Sarafem	500,000
Ibuprofen	Advil, Haltran, Mediprene, Motrin, Nuprin, Proen, Rufen	500,000
Ketamine	Ketalar	1,000,000
LAAM (a-levo-acetylmethadol)		100,000
nor LAAM (a-levo-noracetylmethanol)		100,000
Levothyroxine (T ₄)	Synthroid	500,000
Loratadine	Claritin	500,000
Meperidine	Demerol	1,000,000
Mesoridazine	Serentil	250,000
d-Amphetamine		100,000
l-Methamphetamine		100,000
Methadone		1,000,000
Morphine		1,000,000
Oxazepam	Serax	500,000
Phencyclidine		500,000
Phenobarbital	Luminal	1,000,000
Phentermine	Adipex, Fastin, Ionamin, Obenix, Obephen, Obermine, Obestin, Phentamine,	1,000,000
Promethazine	Phenergan	100,000
Propoxyphene	Darvon	1,000,000
Ranitidine	Zantac	500,000
Salicylic Acid		500,000
Secobarbital	Seconal	1,000,000