

## DRI<sup>®</sup> Opiate Cross-Reactivity Tables Test Code # 509/509Q

### POSITIVE COMPOUNDS

The following compounds tested POSITIVE using the DRI<sup>®</sup> DAU Opiate assay at the 300 mg/dL cutoff.

Positive Compounds	Trade Name	Concentration Tested (ng/mL)
6-Monoacetyl Morphine		280
Codeine		150
Dihydrocodeine	DHC Plus, Synalgos-DC	650
Heroin		380
Hydrocodone	Lortab, Vicoden	650
Hydromorphone	Dilaudid	1,400
Levorphanol	Levo-Dromoran	10,500
Morphine		300
Morphine-3-Glucuronide		340
Morphine-6-Glucuronide		270
Oxycodone	OxyConting	10,500
Oxymorphone	Numorphan	37,000
Ranitidine	Zantac	500,000

### NEGATIVE COMPOUNDS

The following compounds tested NEGATIVE on the DRI<sup>®</sup> DAU Opiate assay at the 300 ng/mL cutoff.

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Acetaminophen	Anacin, Datri Extra, Liquiprin, Panadol Temptra, Tylenol	500,000
Acetylsalicylic Acid	Aspirin, Bufferin	500,000
Acyclovir	Zovirax	100,000
Amitriptyline	Elavil, Endep, Enovil, Emitrip	100,000
Amphetamine		1,000,000
Azithromycin	Zithromax	100,000
Benzoyllecgonine		1,000,000
Buprenorphine	Buprenex	100,000
Bupropion	Wellbutrin, Zyban	100,000
Caffeine	Coffee-Break, Durivitan, No-Doz, Percoffedrinol N, Pro-Plus, Vivarin	100

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Carbamazepine	Carbatrol, Epitol, Tegretol,	500,000
Chlorpromazine	Anafranil	100,000
Clonidine	Catapress	100,000
Cyclazocine		35,000
Cyamemazine		1,000,000
Dapsone	Avlosulfon	10,000
Desipramine	Norpramin, Pertofrane	100,000
o-Desmethylvenlafaxine		1,000,000
Dextromethorphan	Benilyn DM, Delsym, Hold, Pefussin 8 hr, Mediquell, Sucrets	100,000
Doxepin	Adapin, Sinequan	500,000
Doxycycline Hyclate	Periostat	100,000
EDDP (Primary Methadone Metabolite)		62,500
Ephedrine	Kondon's Nasal, Pretz-D	1,000,000
Fentanyl	Duragesic, Actiq, Sublimaze	100,000
Fluconazole	Diflucan	100,000
Fluoxetine	Prozac, Sarafem	100,000
Fluphenazine	Permitil, Prolixin	100,000
Gabapentin	Neurontin	100,000
Hydroxyzine	Anxanil, Apo-Hydroxyzine, Atarax, Hyzine-50	100,000
Hyoscyamine HCl	Anaspaz, Levsin	100,000
Ibuprofen	Advil, Haltran, Mediprene, Motrin, Nuprin, Proen, Rufen	500,000
Imipramine	Janimine, Tofranil	100,000
Lamivudine	Epivir	100,000
Lamotrigene	Lamictal	1,000,000
Levofloxacin	Levaquin	800,000
Loratadine	Claritin	10,000
Maprotiline	Ludiomil	100,000
Meperidine	Demerol	20,000
Methadone	Dolophine	500,000
Metronidazole	Flagyl, Metizol, Protostat	1,000,000
Nalbuphine	Nubain	1,000,000
Naloxone	Narcan	100,000
Naltrexone	Depade, ReVia	3,000,000
Normorphine		100,000
Nortioptiline	Aventyl, Pamelor	100,000

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Oflaxacin	Floxin	100,000
Oxazepam	Serax	250,000
Paroxetine	Paxil	100,000
Pentazocine	Talwin	100,000
Phencyclidine		1,000,000
Phenobarbital	Luminal	1,000,000
Phenytoin (DPH)		500
Propoxyphene	Darvon	900,000
Ranitidine	Zantac	100,000
Rifampin	Rifadin	1,000,000
Risperidone	Risperdal	100,000
Secobarbital	Seconal	1,000,000
Talwin		100,000
Thebaine		2,000
Thiopental	Pentothal Sodium	1,000
Thiorizadine	Mellaril	100,000
Tramadol	Ultram	100,000
Venlafaxine	Effexor	1,000,000