

## DRI<sup>®</sup> Oxycodone Cross-Reactivity Tables Test Code # 177/177Q

### POSITIVE COMPOUNDS

The following compounds tested POSITIVE using the DRI<sup>®</sup> DAU Oxycodone assay at the 100 ng/mL cutoff.

Positive Compounds	Trade Name	Concentration Tested (ng/mL)
Oxycodone	OxyContin	300
Oxymorphone	Numorphan	300
Noroxymorphone		500,000

### NEGATIVE COMPOUNDS

The following compounds tested NEGATIVE on the DRI<sup>®</sup> DAU Oxycodone assay at the 100 ng/mL cutoff.

Negative Compounds	Trade Name	Concentration Tested (µg/mL)
Acetaminophen	Anacin, Datri Extra, Liquiprin, Panadol Temptra, Tylenol	2,000
6-Acetylmorphine		50
Acetylsalicylic Acid	Aspirin, Bufferin	1,000
Amitriptyline	Elavil, Endep, Enovil, Emitrip	500
Amoxicillin	Amoxil, Polymor, Trimox, Wymox	500
Amphetamine		2,000
Benzoylcegonine		2,000
Buprenorphine		10
Caffeine	Coffee Break, Durivitan, No-Doz, Percoffedrinol N, Pro-Plus, Vivarin	1,000
Carbamazepine	Carbatrol, Eptol, Tegretol	1,000
Chlorpromazine	Ormazine, Thorazine	2,000
Cimetidine	Tagamet	1,000
Clomipramine	Anafranil	1,000
Codeine		500
Desipramine	Norpramine, Pertofrane	1,000
Dextromethorpan	Benylin DM, Delsym, Hold, Perfussin 8hr, Mediquell, Sucrets	200

Negative Compounds	Trade Name	Concentration Tested (µg/mL)
Dihydrocodeine		100
Doxepine	Adapin, Sinequan	200
Ephedrine	Kondon's Nasal, Pretz-D	2,000
Fentanyl	Duragesic, Actiq, Sublimaze	200
Fluoxetine	Prozac, Sarafem	1,000
Fluphenazine	Permitil, Proxilin	500
Heroin		300
Hydrocodone	Lortab, Vicodin	75
Hydromorphone	Dilaudid	30
Ibuprofen	Advil, Haltran, Mediprine, Motrin, Nuprin, Proen, Rufen	1,000
Imipramine	Janimine, Tofranil	1,000
Levorphanol		200
Maprotiline	Ludiomil	1,000
Meperidine	Demerol	1,000
Methadone	Dolophine	1,000
Metrodinazole	Flagyl	2,000
Morphine		350
Morphine-3-glucuronide		900
Nalbuphine	Nubain	1,000
Naloxone	Narcan	200
Naltrexone	Depade, ReVia	500
Norcodeine		1,000
Normorphine		1,000
Noroxycodone		50
Nortriptyline	Aventyl, Pamelor	500
Oxazepam	Serax	500
Pentazocine	Talwin	500
Phencyclidine		1,000
Phenobarbital	Luminal	1,000
Ranitidine	Zantac	3,000
Secobarbital	Seconal	1,000
Talwin		500
Thebaine		20
Thiorizadine	Mellaril	1,000
Tramadol	Ultram	500