

DRI[®] Phencyclidine Cross-Reactivity Tables Test Code # 508/508Q

POSITIVE COMPOUNDS

The following compounds tested POSITIVE using the DRI[®] DAU Phencyclidine assay at the 25 ng/mL cutoff.

Positive Compounds	Trade Name	Concentration Tested (ng/mL)
Phencyclidine (PCP)		25
Tramadol Hydrochloride	Ultram	500,000

NEGATIVE COMPOUNDS

The following compounds tested NEGATIVE on the DRI[®] DAU Phencyclidine assay at the 25 ng/mL cutoff.

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Acetaminophen	Anacin, Datri Extra, Liquiprin, Panadol Tempra, Tylenol	1,000,000
Acetylsalicylic Acid	Aspirin, Bufferin	1,000,000
Albuterol	Proventil, Ventolin, Salbutamol	1,000,000
Amikacin	Amikin	1,000,000
Amitriptylline	Elavil, Endep, Enovil, Emitrip	1,000,000
Amobarbital	Amytal	1,000,000
Amoxicillin	Amoxil, Polymor, Trimox, Wymox	1,000,000
Amphetamine		1,000,000
Atropine	Lomotil, Motofen	100,000
Brompheniramine	Dimetane, Dimetapp, Nasahist, ND-Stat, Oraminic II	50,000
Bupropion	Wellbutrin, Zyban	100,000
Calcium Carbonate	Oscal	1,000,000
Calecoxib	Celebrex	1,000,000
Chlorpheniramine	Aller-Chlor, Chlor-Trimeton	50,000
Chlorzoxazone	Paraflex	1,000,000
o-Desmethylvenlafaxine		1,000,000
Dextromethorphan	Benlyn DM, Delsym, Hold, Perfussin 8 hr, Mediquell, Sucrets	1,000,000

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Diphenhydramine	Banophen, Benadryl, Diphedryl, Simply Sleep	100,000
Doxepin	Adapin, Sinequan	500,000
EMDP		100,000
Fluoxetine	Prozac, Sarafem	50,000
Gentamicin	Garamycin	1,000,000
Halperidol	Haldol	1,000,000
Hydroxyzine Dihydrochloride		1,000,000
Ibuprofen	Advil, Haltran, Mediprene, Motrin, Nuprin, Proen, Rufen	5,000,000
Imipramine		500,000
Indomethacin	Indocin, Novomethacin	100,000
Ketamine		100,000
Lamotrigene	Lamictal	1,000,000
Lithium heparin		5,000,000
Loratadine	Claritin	500,000
Meperidine		50,000
Mesoridazine	Serentil	15,625
Methadone	Dolophine	1,000,000
Methaqualone	Normi-Nox, Pallidan, Somnomed, Quaalude	100,000
Methylphenidate	Ritalin	1,500
Metronidazole	Flaglyl, Metizol, Protostat	1,000,000
Morphine	MSIR, Ms Contin, Oramorph Sr, Roxanol, Astramorph, Duramorph, Infumorph	200,000
Nalbuphine	Nubain	1,000,000
Naltrexone	Depade, ReVia	10,000
Naproxen Na ⁺	Aleve, Anaprox, Naprosyn	1,000,000
Norfluoxetine HCl		1,000,000
Omeprazole	Prilosec	1,000,000
Orphenadrine	Flexoject, Myophen, Norflex	200,000
Oxazepam	Serax	100,000
Phencyclidine		400
Phenelzine	Marplan, Nardil, Parnate	100,000
Phenobarbital	Luminal	1,000,000
1-Phenylcyclohexylamine		50,000
1-Piperidinocyclohexane Carbonitrile (PCC)		100,000
Pholcodine		3,900
Promethazine	Phenergan	100,000

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Scopolamine	Transderm-Scop	1,000,000
Spirolactone	Novospiroton	1,000,000
Terbinafide	Lamisil	1,000,000
Thiopental	Pentothal Sodium	1,000,000
Thioridazine	Mellaril	80,000
Tobramycin	Nebcin, Tobi, Tobrex	1,000,000
Tolmetin	Tolectin	1,000,000
Tramadol	Ultram	100,000
Trazodone	Desyrel	1,000,000
Trimethoprim	Proloprim, Trimprex	5,000,000
Tripolidine	Actagen, Allerfrin, Aprodine	10,000
Vancomycin	Vancocin	1,000,000
Venlafaxine	Effexor	1,000,000