

DRI[®] Propoxyphene Cross-Reactivity Tables Test Code # 507/507Q

POSITIVE COMPOUNDS

The following compounds tested POSITIVE using the DRI[®] DAU Propoxyphene assay at the 300 ng/mL cutoff.

Positive Compounds	Trade Name	Concentration Tested (ng/mL)
Propoxyphene		300
Norpropoxyphene		500

NEGATIVE COMPOUNDS

The following compounds tested NEGATIVE on the DRI[®] DAU Propoxyphene assay at the 300 ng/mL cutoff.

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Acetaminophen	Anacin, Datri Extra, Liquiprin, Panadol Temptra, Tylenol	1,000,000
Acetylsalicylic Acid	Aspirin, Bufferin	1,000,000
Albuterol	Proventil, Ventolin, Salbutamol	1,000,000
Amikacin	Amikin	1,000,000
Amitriptylline	Elavil, Endep, Enovil, Emitrip	50,000
Amoxicillin	Amoxil, Polymor, Trimox, Wymox	1,000,000
d-Amphetamine		1,000,000
Atropine	Lomotil, Motofen	100,000
Benzoylcegonine		1,000,000
Bupropion	Wellbutrin, Zyban	100,000
Caffeine	Coffee Break, Durivitan, No-Doz, Percofedrinol N, Pro-Plus, Vivarin	100,000
Calcium Carbonate	Oscal	5,000,000
Carbamazepine	Atretol, Carbatrol, Tegretol	20,000
Chlorpromazine	Thorazine	10,000
Chlorzoxazone	Paraflex	1,000,000
Codeine		500,000
o-Desmethylvenlafaxine		1,000,000
Dextromethorphan	Benylin DM, Delsym, Hold, Perfussin 8 hr, Mediquell, Sucrets	200,000

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Diphenhydramine	Banophen, Benadryl, Diphedryl, Simply Sleep	1,000,000
Doxepin	Adapin, Sinequan	500,000
Doxylamine	Decapryn, Nighttime Sleep Aid, Unisom	100,000
Fluoxetine	Prozac, Sarafem	50,000
Gentamicin	Garamycin	1,000,000
Ibuprofen	Advil, Haltran, Mediprene, Motrin, Nuprin, Proen, Rufen	5,000,000
Indomethacin	Indocin, Novomethacin	100,000
Lamotrigene	Lamictal	1,000,000
Lithium heparin		5,000,000
Loratadine	Claritin	500,000
Mesoridazine	Serentil	1,000,000
Methadone	Dolophine	100,000
Methaqualone	Normi-Nox, Pallidan, Somnomed, Quaalude	500,000
Methylphenidate	Ritalin	100,000
Metronidazole	Flagyl, Metizol, Protostat	1,000,000
Morphine	MSIR, Ms Contin, Oramorph Sr, Roxanol, Astramorph, Duramorph, Infumorph	200,000
Nalbuphine	Nubain	1,000,000
Naltrexone	Depade, ReVia	1,000,000
Naproxen-Sodium	Aleve, Anaprox, Naprosyn	5,000,000
Norfluoxetine HCl		1,000,000
Nortriptyline	Aventyl, Pamelor	50,000
Omeprazole	Prilosec	1,000,000
Oxazepam	Serax	300,000
Phencyclidine		400
Phenelzine	Marplan, Nardil, Parnate	100,000
Pheniramine	Opcon-A	100,000
Phenobarbital	Luminal	1,000,000
Phenytoin	Dilantin	40,000
Pholcodine		3,900
Primidone	Mysoline	24,000
Scopolamine	Transderm-Scop	1,000,000
Secobarbital	Seconal	1,000,000
Spirolactone	Novospiroton	1,000,000
Theophylline	Elixophyllin, Quibron, Respbid, Somophyllin, Slo-Phyllin, Theo-Dur, Slo-Bid, Theo-24, Theolair	40,000

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Thiopental	Pentothal Sodium	1,000,000
Tobramycin	Nebcin, Tobi, Tobrex	1,000,000
Tolmetin	Tolectin	1,000,000
Tramadol	Ultram	500,000
Trazodone	Desyrel	1,000,000
Trimethoprim	Proloprim, Trimprex	5,000,000
Valproic Acid	Depakote	150,000
Vancomycin	Vancocin	1,000,000
Venlafaxine	Effexor	1,000,000